


HOW TO USE A DISPOSABLE SHEATH

1 Camera Insertion


With the lens side down towards the paper backing, slide the camera beneath the white tab into the sheath.

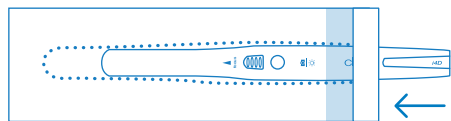

*It is OK to leave a little room at the tip (see below)

2 Protective Covering Removal

Holding the camera at the cable end, pull the blue tab up, removing the protective clear cover, and pull the white paper backing down. Discard covers.

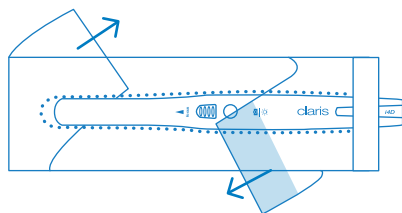


3 Camera Inserted in Sheath




SHEATH TIPS

If the sheath causes difficulty in moving the adjustable focus, simply hold the sheath above the focus area and pull up the portion of the sheath below to give the sheath some slack.

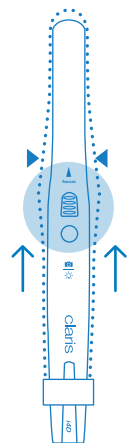


SHEATH TIPS (cont.)


Sometimes, there may be an air pocket between the lens area and sheath, which may cause a glare when capturing images.

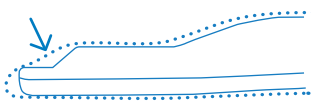

Gently roll your thumb to press the sheath down onto the small black prism box.

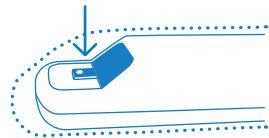


The sheath should stick to the prism box. Be careful not to be too forceful as you can damage the prism.

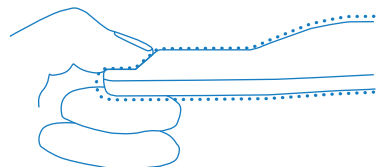

CONNECTING / DISCONNECTING USB CABLE


Firmly press the two buttons on the opposing sides of the USB connector to release the inner latches. While pressing the two buttons, gently wiggle and pull to disconnect the USB.

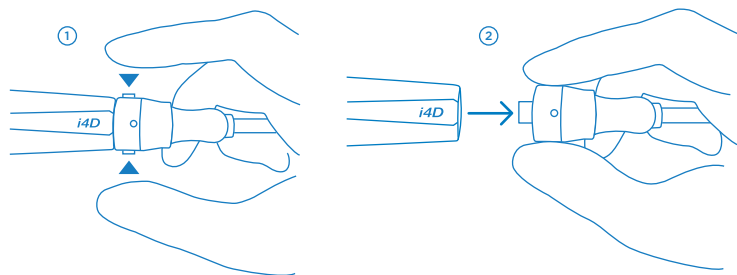



To reduce future wear and tear, press the two buttons when connecting the USB cable as well.

TECHNIQUES

Overhand Grip

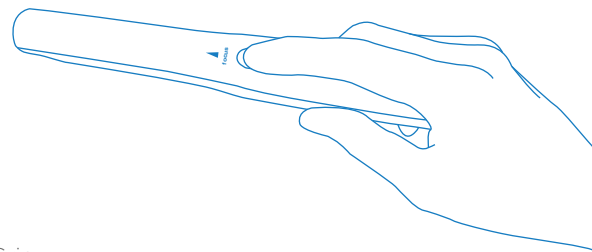


Pencil Grip

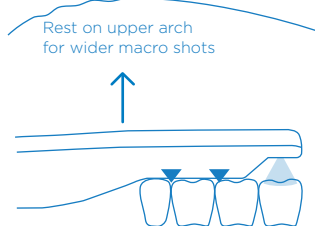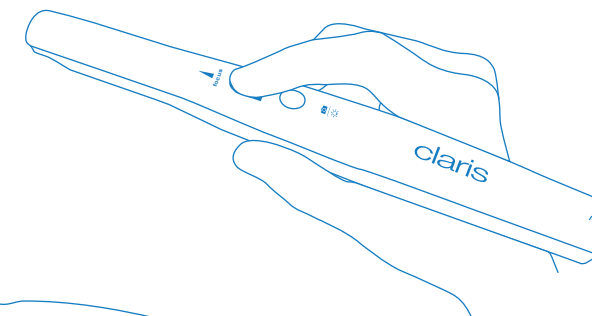

Rest on upper arch for wider macro shots

For close macro shots, rest camera on teeth for image stability. You can also rest top of camera on the upper or opposing arch to acquire stable images of teeth.

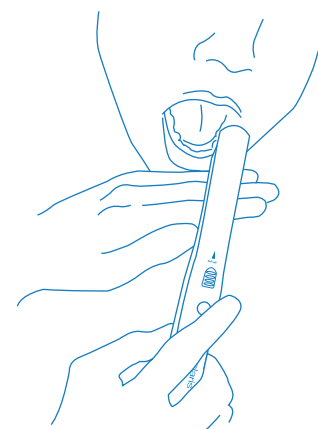

For full smile, bottom arch, and upper arch shots, try placing fingers on patient's chin as shown to steady the camera. Pivot camera up or down to achieve desired angle.

FOCUS CONTROL AND CAPTURE

All-the-way Up (Auto-focus):


full face, full smile, full arch, arch quadrant

Half-way Down:

1 to 2 molar shot

All-the-way Down:

1/2 molar shot

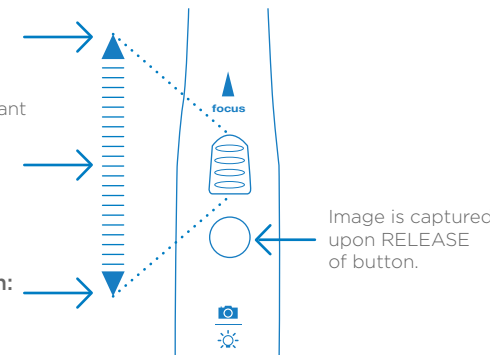

TURNING CAMERA OFF

1 Mount camera cradle by securing the flat back side to wall or surface. Make sure the diagonal sloped side is facing upwards as shown.


Diagonal sloped side should face up



1. Mount flat back side to surface with screws or double-sided tape

2 Place camera bottom (the side where the USB connector sits) through the cradle top-opening.

The face of the camera with the capture button, focus, and Claris logo should be facing OUT towards you. There is a magnetized component within the cradle which will shut the camera OFF.

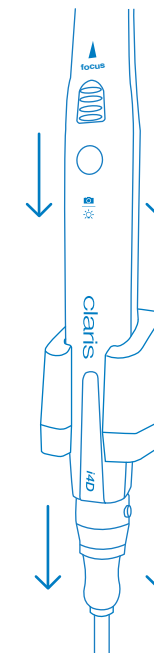

MAINTENANCE

- 1 Clean lens/prism (small black square in center of LED lights) with alcohol and Q-tip at least once a week but preferably everyday before use. This will ensure a clear image.
- 2 DO NOT have patient bite down on camera head. Biting down may cause the lens or surrounding glass to chip or break.